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Specific Test Parameters	
General	LIH ISE Range
Test Name:	CysC < > Type: Serum Operation: Yes
Sample:	Volume: 2 μL Dilution: 0 μL Pre-Dilution Rate: 1
Reagents:	R1 Volume: 150 μL Dilution: 0 μL Min OD: Max OD
	R2 Volume: 30 μL Dilution: 10 μL Reagent OD limit: L H
Wavelength:	Pri. 540 Sec. First L: -2.0 First H: 2.0
Method:	End Point Last L: Last H:
Reaction slope:	+ Dynamic Range: L * H *
Measuring Point 1:	First 13 Last 27 Correlation Factor: A 1.00 B 0.00
Measuring Point 2:	First Last
Linearity:	% On-board stability period: 60
No Lag Time:	

Specific Test Parameters	
General	LIH ISE Range
Test Name:	CysC < > Type: Serum
Value/Flag:	# Level L: # Level H: #
Normal Ranges:	Age L Age H
	Sex Year Month Year Month L H
o 1.	# # # # # # # #
o 2.	# # # # # # # #
o 3.	# # # # # # # #
o 4.	# # # # # # # #
o 5.	# # # # # # # #
o 6.	# # # # # # # #
7.	None Selected
8.	Out of Range L H
Panic Value:	# # Unit: mg/L Decimal places: #

Calibration Specific	
General	ISE
Test Name:	CysC < > Type: Serum
Calibration Type:	6AB Formula: Spline Counts: # Process: CONC
Point 1:	Cal. No. 1 OD CONC ** Factor/OD-L Factor/OD-H
Point 2:	2
Point 3:	3
Point 4:	4
Point 5:	5
Point 6:	6
Point 7:	
1-Point Cal. Point:	o with CONC-0 Slope Check: + Advanced Calibration: #
MB Type Factor:	Calibration Stability Period: 28

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Parameters		Specific Test Parameters			
General	LIH	ISE	Calculated Test	Range	
Test Name: <input type="text" value="CysC"/> < > Type: <input type="text" value="Serum"/> Operation: <input type="text" value="Yes"/>					
Sample Volume	<input type="text" value="2"/> μL	Dilution	<input type="text" value="0"/> μL	OD Limit	
Pre-Dilution Rate	<input type="text" value="1"/>			Min.OD	<input type="text" value=""/>
Rgt. Volume	R1(R1-1) <input type="text" value="150"/> μL	Dilution	<input type="text" value="0"/> μL	Reagent OD Limit	
				First	Low <input type="text" value="-2.0"/> High <input type="text" value="2.0"/>
				Last	Low <input type="text" value=""/> High <input type="text" value=""/>
	R2(R2-1) <input type="text" value="30"/> μL	Dilution	<input type="text" value="10"/> μL	Dynamic Range Low <input type="text" value=""/> High <input type="text" value=""/>	
Common Rgt. Type	<input type="text" value=""/>	Name	<input type="text" value=""/>	Correlation Factor A	<input type="text" value="1.00"/> B <input type="text" value="0.00"/>
Wavelength	Pri <input type="text" value="540"/> nm	Sec.	<input type="text" value=""/>	Factor for Maker A <input type="text" value=""/> B <input type="text" value=""/>	
Method	<input type="text" value="End Point"/>				
Reaction Slope	<input type="text" value="+"/>				
Measuring Point1 First	<input type="text" value="13"/>	Last	<input type="text" value="27"/>	Onboard Stability Period	<input type="text" value="60"/> Day <input type="text" value=""/> Hour
Measuring Point2 First	<input type="text" value=""/>	Last	<input type="text" value=""/>	LIH Influence Check	<input type="text" value=""/>
Linearity Limit	<input type="text" value=""/>				
Lag Time Check	<input type="text" value=""/>				
				Lipemia	<input type="text" value=""/>
				Icterus	<input type="text" value=""/>
				Hemolysis	<input type="text" value=""/>

Parameters		Specific Test Parameters			
General	LIH	ISE	Calculated Test	Range	
Test Name: <input type="text" value="CysC"/> < > Type: <input type="text" value="Serum"/>					
Value/Flag:	<input type="text" value="#"/>	Low	<input type="text" value=""/>	High	<input type="text" value=""/>
Level	<input type="text" value=""/>				
Specific Ranges:	From	To		Low	High
Sex	Year	Month	Year	Month	Low
1. <input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>
2. <input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>
3. <input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>
4. <input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>
5. <input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>
6. <input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>
7. No demographics	<input type="text" value=""/>				
8. Not within expected values	<input type="text" value=""/>				
Unit	<input type="text" value="mg/L"/>	Decimal Places	<input type="text" value="#"/>		
				Panic Value	Low <input type="text" value=""/> High <input type="text" value=""/>

Parameters		Calibration Parameters			
Calibrators	Calibration Specific	STAT Table Calibration			
General	ISE				
Test Name: <input type="text" value="CysC"/> < > Type: <input type="text" value="Serum"/> <input type="checkbox"/> Use Serum Cal.					
Calibration Type:	<input type="text" value="6AB"/>	Formula:	<input type="text" value="Spline"/>		Counts:
<Calibrator Parameters>					
Calibrator	OD	Conc	Factor Range		Slope Check
Point 1:	<input type="text" value="1"/>	<input type="text" value="**"/>	<input type="text" value=""/>		<input type="text" value="+"/>
Point 2:	<input type="text" value="2"/>	<input type="text" value="**"/>	<input type="text" value=""/>		Allowance Range Check
Point 3:	<input type="text" value="3"/>	<input type="text" value="**"/>	<input type="text" value=""/>		<input type="checkbox"/> Reagent Blank <input type="text" value=""/>
Point 4:	<input type="text" value="4"/>	<input type="text" value="**"/>	<input type="text" value=""/>		<input type="checkbox"/> Calibration <input type="text" value=""/>
Point 5:	<input type="text" value="5"/>	<input type="text" value="**"/>	<input type="text" value=""/>		Advanced Calibration
Point 6:	<input type="text" value="6"/>	<input type="text" value="**"/>	<input type="text" value=""/>		Operation <input type="text" value="#"/>
Point 7:	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>		Interval (RB/ACAL) <input type="text" value="#"/>
Point 8:	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>		
Point 9:	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>		
Point 10:	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>		
<Point Cal. For	No. of Correction Points <input type="text" value=""/>	Use Master Curve <input type="checkbox"/>	<input type="checkbox"/> Lot Calibration		
Master Curve>					
Calibrator	OD	Conc	Low High		Stability
Point-1	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>		Reagent Blank <input type="text" value="28"/> Day <input type="text" value=""/> Hour
Point-2	<input type="text" value=""/>	<input type="text" value=""/>	<input type="text" value=""/>		Calibration <input type="text" value="28"/> Day <input type="text" value=""/> Hour
MB Type Factor:	<input type="text" value="1-Point Calibration Point"/>		<input type="checkbox"/> with Conc-0		

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Parameters		Specific Test Parameters			
General	LIH	ISE	Calculated Test	Range	
Test Name: <input type="text" value="CysC"/> < > Type: <input type="text" value="Serum"/> Operation <input type="text" value="Yes"/>					
Sample Volume	<input type="text" value="2"/> μL	Dilution	<input type="text" value="0"/> μL	OD Limit	
Pre-Dilution Rate	<input type="text" value="1"/>	Diluent Bottle	<input type="text" value="Outside"/>	Min.OD	Max.OD
Rgt. Volume	<input type="text" value="150"/> μL	Dilution	<input type="text" value="0"/> μL	Reagent OD Limit	
R1-2	<input type="text" value=""/>	Dilution	<input type="text" value=""/>	First	Low
				Last	Low
R2(R2-1)	<input type="text" value="30"/> μL	Dilution	<input type="text" value="10"/> μL	High	<input type="text" value="2.0"/>
Common Rgt. Type	<input type="text" value=""/>	Name	<input type="text" value=""/>	Dynamic Range Low	High
Wavelength	<input type="text" value="540"/> nm	Sec.	<input type="text" value=""/>	Correlation Factor A	B
Method	<input type="text" value="End Point"/>			Factor for Maker A	B
Reaction Slope	<input type="text" value="13"/>	Last	<input type="text" value="27"/>	Onboard Stability Period	<input type="text" value="60"/> Day <input type="text" value=""/> Hour
Measuring Point1 First	<input type="text" value=""/>	Last	<input type="text" value=""/>	LIH Influence Check	<input type="text" value=""/>
Measuring Point2 First	<input type="text" value=""/>			Lipemia	<input type="text" value=""/>
Linearity Limit	<input type="text" value=""/>			Icterus	<input type="text" value=""/>
Lag Time Check	<input type="text" value=""/>			Hemolysis	<input type="text" value=""/>

Parameters		Specific Test Parameters			
General	LIH	ISE	Calculated Test	Range	
Test Name: <input type="text" value="CysC"/> < > Type: <input type="text" value="Serum"/>					
Value/Flag: <input type="text" value="#"/>					
Level					
Level Low <input type="text" value="#"/> High <input type="text" value="#"/>					
Specific Ranges: From To Low High					
	Sex	Year	Month	Year	Month
1.	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>
2.	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>
3.	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>
4.	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>
5.	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>
6.	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>	<input type="text" value="#"/>
7.	Standard demographics				
8.	Not within expected values				
Panic Value	Low	High	Unit	mg/L	Decimal Places <input type="text" value="#"/>

Parameters		Calibration Parameters			
Calibrators	Calibration Specific	STAT Table Calibration			
General	ISE				
Test Name: <input type="text" value="CysC"/> < > Type: <input type="text" value="Serum"/> Cuvette: <input type="text" value=""/>					
<input type="checkbox"/> Use Serum Cal.					
Calibration Type: <input type="text" value="6AB"/> Formula: <input type="text" value="Spline"/> Counts: <input type="text" value="#"/>					
<Calibrator Parameters>					
Point 1:	<input type="text" value="1"/>	OD	<input type="text" value="**"/>	Factor Range	
Point 2:	<input type="text" value="2"/>		<input type="text" value="**"/>	Low	High
Point 3:	<input type="text" value="3"/>		<input type="text" value="**"/>		
Point 4:	<input type="text" value="4"/>		<input type="text" value="**"/>		
Point 5:	<input type="text" value="5"/>		<input type="text" value="**"/>		
Point 6:	<input type="text" value="6"/>		<input type="text" value="**"/>		
Point 7:	<input type="text" value=""/>		<input type="text" value=""/>		
Point 8:	<input type="text" value=""/>		<input type="text" value=""/>		
Point 9:	<input type="text" value=""/>		<input type="text" value=""/>		
Point 10:	<input type="text" value=""/>		<input type="text" value=""/>		
Slope Check <input type="text" value="+"/> <input type="text" value=""/>					
Allowance Range Check					
<input type="checkbox"/> Reagent Blank <input type="text" value=""/>					
<input type="checkbox"/> Calibration <input type="text" value=""/>					
Advanced Calibration					
Operation <input type="text" value="#"/> <input type="text" value=""/>					
Interval (RB/ACAL) <input type="text" value="#"/> <input type="text" value=""/>					
<Point Cal. For No. of Correction Points <input type="text" value=""/> Use Master Curve <input type="text" value=""/> Lot Calibration <input type="text" value=""/>					
Master Curve>					
Point-1	<input type="text" value=""/>	OD	<input type="text" value=""/>	Low	High
Point-2	<input type="text" value=""/>		<input type="text" value=""/>		
Stability					
Reagent Blank <input type="text" value="28"/> Day <input type="text" value="0"/> Hour					
Calibration <input type="text" value="28"/> Day <input type="text" value="0"/> Hour					
MB Type Factor: <input type="text" value=""/> 1-Point Calibration Point <input type="text" value=""/> <input type="checkbox"/> with Conc-0					

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